Gebrauchsinformation »Retraction paste 2«

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Handelsform

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Instructions for Use "*Retraction paste 2*"

Retraction paste 2 is a retraction agent for sulcus expansion. *Retraction paste 2* has an astringent effect on the gingiva.

Indication

For temporary marginal gingival retraction and drying of the sulcus in the case of healthy periodontium.

Contraindications

Retraction paste 2 contains kaolin and aluminum chloride. Do not use the paste in patients with hypersensitivity to kaolin and aluminum chloride.

Do not use in the case of exposed bone, advanced periodontal disease or exposed furcations.

Side effects

There are no known side effects to date. Isolated cases of hypersensitivity to components of the material cannot be excluded.

Basic safety instructions

- For dental use only!
- Keep away from children!
- Avoid contact with eyes! In case of accidental contact with eyes, rinse immediately with plenty of water and consult a doctor if necessary.
- Do not swallow the paste! In case of accidental swallowing, consult a doctor if necessary.

Recommended use

1. Prepare the tooth as needed.
2. Place compule in a suitable dispenser.
3. Bend the tip of the compule into the required position if necessary.
4. Extrude a small amount of the paste onto a mixing palette and discard.
5. Optional: If required by the clinical situation, place a retraction thread in the sulcus.
6. Rinse the sulcus with water and dry.
7. Apply paste slowly in the sulcus around the preparation until the sulcus is completely filled. Use compression caps, compression protection or other compression products as needed.
8. Allow paste to work for approx. 2 minutes in order to enable gingival retraction. During the working time, the area treated must be kept absolutely dry.

Caution: Leaving the paste in the mouth too long may lead to irritation of the gingiva. Do not leave in the mouth longer than recommended.

9. Completely remove paste from the sulcus and mouth area using a water/air mixture and suction device.

Note: Remove paste completely.

10. Paste residues can cause inaccuracies or delays in setting of the impression material. Dry gingiva with air and check the retraction result. If greater retraction is desired, repeat procedure from step 7. If necessary, additionally use a thin retraction thread.

11. Create impression.

Technical data

Working time = 2 min

Composition

Kaolin, aluminum chloride hexahydrate, water, pigments, flavoring

Storage

- Store at temperatures between 15 °C – 25 °C / 59 °F – 77 °F.
- Do not use after the expiration date.
- After opening the aluminum pouch, use compule immediately. Dispose of remaining material in the compule.

Available Product Sizes

REF 9119

5 compules @ 0.37 g