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Instructions for use »VPS Bite scan«

Keep away from children! For dental use only!

VPS Bite scan is a reflection active material for powderfree optical registration of data (antagonist registration) for CAD/CAM/CIM based on addition-cured silicone.

Indication

- Bite registration
- Powderfree optical registration of data (antagonist registration) for CAD/CAM/CIM
- Material for application on bite forks used with face-bows
- Key material, e.g. for intraoral incisal pin records

Cartridge handling

The material is dispensed and mixed automatically by extruding through a mixing tip. Remove the transport seal before using a new cartridge. Then attach the mixing tip and, if required, a conturation tip. Following use, the mixing tip should be used as a seal and can be left on the cartridge until the next application. The mixing tip should only be replaced with a new one when the cartridge is to be used again.

Recommended use

1. Apply the material directly onto the occlusal surfaces. Cover prepared areas first.
2. Usually, only the teeth of the upper or lower arch need to be covered. When recording anterior teeth, it is suggested to place material over the teeth of both arches. A layer of about 5 mm usually is sufficient.
3. Let the patient close into desired occlusion.
4. Remove set material after 1:15 minutes carefully.
5. Due to its high final hardness, the material can be finished with a rotating instrument (e.g. mill). Directly after removal from mouth excess can be removed with a scalpel. Cut or mill the registrate mesial and distal.
6. Place registrate back onto preparation and register it optically.

Disinfection

Disinfection with standard disinfection solutions.

Please note

Do not use with condensation-cured or polyether impression materials.

Avoid contact of the impression with organic solvents or liquids containing solvent, as this can cause the material to swell and lead to inaccuracies.

Retraction solutions should be thoroughly sprayed and dried before taking the impression to avoid them impairing the setting properties. Do not use with latex gloves.

In rare cases it cannot be ruled out that predisposed patients may develop hypersensitivity to individual components of the material.

Technical data

Working time (incl. mixing time) ≤ 0:30 min

Recommended time in mouth ≥ 1:15 min

Shore-A hardness (after 24 h) ≥ 92

Times refer to normal room temperature (23 °C / 73 °F) and a normal relative humidity (50%). Higher temperatures shorten, lower temperatures increase those times.

Composition

Viny silicones/siloxanes, hydro silicones/siloxanes, inorganic filler, pigments, additives, catalyst

Storage

Store at normal room temperature (15-25 °C / 59-77 °F). Do not use after date of expiry.

Available product sizes

VPS Bite scan

REF 9024 2 Cartridges @ 50 ml

12 Mixing tips