»Soft relining varnish«

de

Produktbeschreibung

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en-US

Product description

Soft relining varnish is a two-component protective varnish for use with permanently soft relining materials based on addition curing silicones (e.g. *VPS Soft relining material*).

Indications

Atrophy of the jaw and sharp alveolar ridge edges

Contraindications

Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

Surface sealant for permanently soft relining materials based on addition curing silicones.

Limitations of use

There are no known limitations of use.

Intended users

Dentist, dental technician

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

- Using a 1:1 ratio, homogeneously mix the *Soft relining varnish* Base and Kat together for 30 seconds.
- Using a brush, apply the mixed material as bubble free as possible onto the soft relining material within the working time specified under "Timing".
- Allow *Soft relining varnish* to dry for 15 min.

Timing

Mixing time	30 s
Working time at room temperature	≤ 1.5 min
Hardening time at room temperature	≥ 15 min

Notes for use

The recommended times can be found in the section headed "Timing"

Interactions

- It is important to ensure that the surface of the soft relining material does not become contaminated by moisture, saliva, blood or monomer-containing resins before applying the *Soft relining varnish*.
- Avoid contact between *Soft relining varnish* and latex gloves. The use of PVC or PE-based gloves is recommended.

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

Working time ≤ 90 s (including 30 s mixing time), hardening time at 23°C / 73°F ≤ 15 min

Storage / disposal

- Store in a dry place at 15 to 25°C (59 to 77°F)! Do not store in the refrigerator or freezer!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Vinyl silicone (approx. 86 %), hydrogen silicone (approx. 13 %), additives

Packaging

REF 9092MDR	1 Bottle Base @ 10 ml
	1 Bottle Kat @ 10 ml