## Gebrauchsinformation »Adhesive for composite«

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 0482

## Instructions for use »Adhesive for composite«

*Adhesive for composite* is a light-curing bonding agent for intraoral repair of damaged composite restorations. *Adhesive for composite* ensures the bond of new composite material applied to cured, older composites. Complete removal of the restoration is no longer required due to intraoral application.

### Indication

Intraoral repair of damaged composites, shade adjustment of aesthetically unsatisfactory composite restorations.

### Recommended application

1. Roughen the area of the restoration to be repaired using a coarse grit abrasive disc or diamond rotary instrument ( $R_a$  approx. 150  $\mu\text{m}$ ) and round off the margins of the fracture. The use of a rubber dam is recommended.
2. Etch exposed enamel surfaces with 37% phosphoric acid gel (e.g. *Etching gel*) before applying *Adhesive for composite*. Adhere strictly to the manufacturer's instructions for the etch gel.  
  
Exposed dentine surface must be conditioned with an adhesive system (e.g. *Self etching adhesive*). Adhere strictly to the manufacturer's instructions for the adhesive system.
3. Apply one drop of *Adhesive for composite* to the repair surface using a brush and rub in for a minimum of 30 s. This time should be increased to approx. 60 s when repairing chemically cured composites.

**Important! Adhere strictly to these times to ensure that the adhesive can bond properly with the restoration material.**

4. Gently air dry the material with oil-free and water-free air, so that a slightly moist layer remains on the surface.
5. Then light cure the *Adhesive for composite* for a minimum of 20 s. We recommend a light-curing time of 30 s when using *Adhesive for composite* with chemically cured composites.
6. Apply the restorative material (e.g. *Nano hybrid composite*) according to the manufacturer's instructions.

### Please note

- Do not apply *Adhesive for composite* to exposed dentin surfaces.
- Light units should have a light intensity of 400 mW/cm<sup>2</sup>, an emission of 450 nm and the performance should be regularly checked. The light should be placed as near as possible to the material.
- The oxygen inhibition layer produced during light curing is used for bonding with materials applied over the layer and should therefore not be removed.
- Avoid contamination of the contents of the bottle caused by dirty single-use brushes and premature polymerization due to ambient light.

### Interactions

*Adhesive for composite* should not be used in conjunction with materials containing eugenol, as they could inhibit polymerization.

### Contraindications

Application of *Adhesive for composite* is contraindicated, if the treatment site cannot be reliably isolated during application, if the pulp or dentin is exposed, or if there is a known allergy to one of the ingredients of the material.

### Side effects

Side effects are not known to date. In singular cases, hypersensitivity to components of the material cannot be excluded.

### Composition

Bis-GMA based resin matrix, additives, catalyst.

### Storage

Store in a dark and dry place at normal room temperature (15-25 °C / 59-77 °F). Do not use after date of expiry.

## Available product sizes

### Adhesive for composite

REF 2051 1 Bottle @ 5 ml

10 Single-use brushes

1 Mixing pallet

**Keep away from children! For dental use only!**