Gebrauchsinformation » *Bis-acrylic temp. crown & bridge mat 1:1*«

Produktbeschreibung

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Instructions for use » *Bis-acrylic temp. crown & bridge mat 1:1*«

Product description

Bis-acrylic temp. crown & bridge mat 1:1 is a bis-acrylic based temporary crown and bridge material that is free from methyl methacrylate. The material is automatically mixed in a ratio of 1:1. Thanks to its excellent mechanical values, *Bis-acrylic temp. crown & bridge mat 1:1* is extremely fracture-resistant and thus most suitable for fabricating long-term temporary restorations. *Bis-acrylic temp. crown & bridge mat 1:1* guarantees a highly accurate fit of the finished temporary restoration with optimal marginal fit.

Indications

- Extensive tooth defects: Tooth decay, severe wear, traumatic injuries of the dental enamel
- Shape and positional anomalies of the teeth
- Tooth loss

Contraindications

Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

Fabrication of temporary crowns, bridges, inlays, onlays, partial crowns, veneers and long-term temporaries

Limitations of use

Use the material to create temporary restorations only

Intended users

Dentist, dental assistant staff and dental technicians

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

A dental impression should be taken using alginate or silicone before preparing a stump for a crown or bridge or before a planned extraction. The interdental flash of the impression should be cut away to improve the stability of the temporary restoration to be fabricated at a later stage. When using alginate impressions, apply the *Bis-acrylic temp. crown & bridge mat 1:1* as soon as possible after taking the impression.

Recommended use and elastic phase

1. Dry the prepared teeth and lightly coat the stumps, surrounding tissue and any existing synthetic material with a separating agent (e.g. Vaseline).
2. Apply *Bis-acrylic temp. crown & bridge mat 1:1* into the deepest section of the impression and then fill up to the gingival areas. Always keep the mixing tip immersed in the material to avoid air bubbles.
3. Reposition the filled impression in the mouth within 30 seconds.
4. *Bis-acrylic temp. crown & bridge mat 1:1* has a firm, elastic consistency approximately 01:45 to 02:15 min after mixing and can be easily removed from the patient's mouth. The curing process must be monitored intra-orally (e.g. with a probe), as the temporary restoration can only be easily removed during the elastic phase.
5. The final chemical curing of *Bis-acrylic temp. crown & bridge mat 1:1* occurs within 04:00 min after mixing. To this end, the temporary restoration should be placed back into the impression. After the removal of the oxygen-inhibited layer with a solvent (e.g. alcohol), the temporary restoration can be refined and polished. For a shiny surface sealing, Varnish & Bond can be applied to the entire temporary restoration and light-cured (according to the manufacturer's instructions).
6. Commercially available temporary cements (e.g. *Temporary luting cement*) can be used for luting the temporary restoration.

Repair of temporary restorations

A feature of temporary restorations fabricated using *Bis-acrylic temp. crown & bridge mat 1:1* is their high mechanical strength. If a temporary restoration should fracture, however, we recommend the following procedure:

Fracture of the temporary restoration shortly after fabrication

Bond the fractures with freshly mixed *Bis-acrylic temp. crown & bridge mat 1:1*.

Press the fragments together for approx. 03:00 min. After *Bis-acrylic temp. crown & bridge mat 1:1* is completely chemically cured, the temporary restoration can be finished as usual.

Fracture of an older or in situ temporary restoration

Roughen the fractured sections and prepare undercuts.

Bond the prepared fracture with freshly mixed *Bis-acrylic temporary crown & bridge material 1:1*. We recommend using a composite bonding agent (e.g. Varnish & Bond) to improve the bond.

Press the fragments together for approx. 03:00 min.

After *Bis-acrylic temp. crown & bridge mat 1:1* is completely chemically cured, the temporary restoration can be finished as usual.

Timing

00:00 min	Injection into the impression
00-30 s	Insert the impression into the mouth
01:45-02:15 min	Removal from the mouth
04:00 min	Complete chemical self-curing

Notes for use

If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.

Interactions

- Residues of methacrylate-based materials may alter the setting behavior of the silicone impression and bite registration materials.
- Materials containing eugenol, moisture and oily air may inhibit polymerization at the contact area.

Cartridge handling

The material is dosed and mixed automatically by being extruded through a mixing tip. When using a new cartridge, remove the transport cap first. The mixing tip can then be fitted. After use and until the next treatment, leave the mixing tip on the cartridge as a seal. The used mixing tip is replaced with a new tip when the device is used for the next treatment.

Note: When using a cartridge for the first time, discard a small amount of the initially ejected material (e.g. a pea-sized amount).

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings / precautions

- For dental use only!
- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

Compressive strength: ≥ 250 MPa, flexural strength ≥ 75 MPa

Storage / disposal

- Store in a dry place at 2 to 25°C/36 to 77°F! Ideally, the unopened material should be kept in a refrigerator.
- Do not use after the expiration date.
- Disposal must comply with national regulations.

Composition

Dental Glass, DMA, EBPADMA, Bis-GMA, TEDMA, additives.

Packaging

REF 1108 1 Cartridge (A2) @ 50 ml

15 Mixing-Tips