»Temporary composite luting cement transparent«

de

Produktbeschreibung

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en-US

Product description

Temporary composite luting cement transparent is a transparent, dual-cure composite luting cement that is indicated for temporary luting of temporary crowns, bridges, veneers, inlays, and onlays. *Temporary composite luting cement transparent* is automatically mixed in the double-barreled syringe and applied directly.

Indications

- Extensive tooth defects: Tooth decay, severe wear, traumatic injuries of the dental enamel
- Tooth loss

Contraindications

Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

- Temporary luting of temporary crowns, bridges, inlays and onlays and temporary veneers.
- Temporary luting of implant-supported temporary prosthesis.

Limitations of use

Do not apply the material directly to the exposed dental pulp.

Intended users

Dentist

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

1. If required, apply a separating agent to the tooth core, core build-up, or temporary (see also Notes for use).
2. Apply the material to the temporary restoration.
3. Insert the restoration within 1:00 minute after mixing has begun and cement it to the dry, prepared tooth using slight pressure.
4. If necessary, instruct the patient to adopt the habitual occlusion.
5. Remove excess cement within 1:15 to 2:00 min of seating the restoration with the aid of a microbrush, brush, foam pellet, or scaler. Drying with oil-free and water-free air makes excess cement easier to identify. Remove interproximal excess by flossing carefully. It is alternatively possible to remove excess cement by briefly curing it using light (1-2 seconds per side) and then detaching it with a scaler.
6. Allow the material to chemically cure completely for 6:00 to 7:00 min after mixing has begun.
7. Translucent restorations: Additionally cure the material with a suitable light-curing unit for at least 20 seconds per side, as required.

Timing

Working time (including mixing time)	1:00 min
Removal of excess material (without light-curing).	1:15–2:00 min after insertion of the restoration
Optional: Light-curing time for removal of excess material (per side)	1–2 s
Intraoral curing	6:00 – 7:00 min
Light-curing (per side)	20 s

Notes for use

- Light units should have a luminous intensity of 400 mW/cm², emit at 450 nm, and their performance should be checked regularly. The light-curing unit should be positioned as closely to the material as possible. Observe the manufacturer's instructions.
- The material adheres to composites. Apply a thin layer of a separating agent (e.g. vaseline, K-Y Jelly) to existing composite-based fillings, underfillings or build-up fillings before using the material. In the case of further use of the temporary, apply a separating agent in advance.
- The removal of the cement from the restoration is made easier by ultrasound.

Interactions

Not be used in conjunction with materials that contain eugenol / clove oil as these inhibit the hardening of the materials.

Using the double-barreled syringe

The double-barreled syringe enables automatic dispensing and mixing of the material, which allows a time-saving direct application. When using a new double-barreled syringe, remove the transport cap first. The mixing tip can then be fitted. After use and until the next treatment, leave the mixing tip on the syringe as a seal. The used mixing tip is replaced with a new tip when the device is used for the next treatment.

Note: When using a double-barreled syringe for the first time, discard a small amount of the initially ejected material (e.g. a pea-sized amount).

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.

»Temporary composite luting cement transparent«

- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

Compressive strength \leq 150 MPa

Storage / disposal

- Store in a dry place at room temperature (15-25°C/59-77°F)!
- Protect against exposure to direct sunlight!
- After application, leave the used mixing tip on the syringe as a cap!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Resin matrix based on EBPADMA, UDMA, PE resin, TEDMA, HEMA (approx. 59 %), dental glass/silica (approx. 40 %), additives.

Packaging

REF 2092MDR	1 double-barreled syringe, 5 ml, 10 mixing tips
-------------	--