## »VPS Situation«

de

### Produktbeschreibung

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en-US

### Product description

*VPS Situation* is a medium consistency precision impression material based on VPS addition-curing silicones and is automatically mixed. The material is suitable for all kinds of anatomical impressions.

### Indications

Dental cases that require the production of situation models.

### Contraindications

Do not use the material where there are contact allergies or known allergies to any of the components.

### Intended use

- Impressions with fully dentulous, partially dentulous or edentulous jaws for making situation models
- Impressions for the fabrication of temporary crowns and bridges
- Impressions for the fabrication of thermoformed splints
- Orthodontic work
- Opposing jaw models

### Limitations of use

There are no known limitations of use.

### Intended users

Dentist, dental assistant staff

### Patient target group

Persons receiving treatment as part of a dental procedure.

### Recommended use

#### Monophase impressions

Load the impression tray with *VPS Situation* and place the tray into the patient's mouth. This position should be maintained for at least as long as the specified setting time in the mouth for the material. After taking the impression, clean the impression thoroughly and rinse under running water for approx. 15 s. Dry the impression carefully to obtain a bubble-free model.

#### Disinfection and casting

The impression can be disinfected using commercially available dipping solutions. The use of a 2 % glutaraldehyde solution is recommended. Please ensure that the disinfectant used is indicated for use with addition-cured silicone impression materials.

The exposure period for the impression is approximately 10 to 15 min. Then rinse under running water for 15 s. Store the impression at normal room temperature (max. 25 °C/77 °F). Wait at least 30 min after removing the impression from the patient's mouth before casting.\*

The impression material is compatible with all commercially available plasters. The use of a hard plaster (ISO 6873:2013, Type 3, e.g. Moldano, Bayer) for the cast and an ultra-hard dental plaster for the molds (ISO 6873:2013, Type 4, e.g. Fujirock II, GC) is recommended. The impressions can be copper or silver-plated.

\* Additional information: Statement by the DGZMK 3/93 V 1.0; yrym.dgzmk.de  
ADA recommends following the advice of the Council on Dental Materials, Instruments and Equipment. Disinfection of impressions.  
JADA1991;122(3):110; wym.ada.org

### Storing the impression

- Store the impression at 15–25 °C (59–77 °F) in a dry place!
- Protect against exposure to direct sunlight!
- The impression is stable and can be cast even after months without loss of accuracy.

### Timing

Working time (from the start of mixing)*	≤ 75 s
Recommended setting time in the mouth	≥ 105 s

\* The specified working times are valid for a room temperature of 23 °C (72 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times.  
The working time describes the period after mixing has begun until the time by which the impression tray must be positioned in the patient's mouth

### Notes for use

- The recommended times can be found in the section headed »Timing«.
- If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- Perforated or non-perforated metal or plastic trays can be used with *VPS Situation*.
- When selecting the impression tray, ensure that there is enough room between the wall of the impression tray and the dental arch.
- In order to improve the adhesion of *VPS Situation* to non-perforated impression trays, apply a thin, even film of *Tray adhesive for VPS*. Allow *Tray adhesive for VPS* to dry for 3 min before loading the impression tray.

### Interactions

- Do not use with condensation-curing or polyether impression materials.
- Do not allow the impression to come into contact with organic solvents or liquids that contain solvents as this can cause the material to swell and lead to inaccuracies.
- As retraction solutions may impair the setting time of the material, any retraction solution residues must be removed prior to taking the impression by thoroughly spraying and drying the region.
- Do not use with latex gloves because they inhibit the setting process. The use of PVC or PE-based gloves is recommended.

### Cartridge handling

The double syringes and cartridges enable the automatic dispensing and mixing, which allows a time-saving direct application. When using a new cartridge, remove the transport cap first. The mixing tip can then be fitted. After use and until the next treatment, leave the mixing tip on the cartridge as a seal. The used mixing tip is replaced with a new tip when the device is used for the next treatment.

## »VPS Situation«

---

**Note:** When using a double syringe or cartridge for the first time, discard a small amount of the initially ejected material (e.g. a pea-sized amount).

---

### Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

### Warnings/precautions

- Keep out of the reach of children!
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

### Product performance characteristics

- The product complies with ISO 4823:2021, medium-bodied consistency
- Mixing ratio of base/catalyst 4:1

### Storage/disposal

- Store in a dry place at 15 to 25 °C (59 to 77 °F).
- Do not use after the expiration date.
- Disposal must comply with national regulations.

### Composition

SiO<sub>2</sub>, vinyl silicones/siloxanes, methyl silicones/siloxanes, hydrogen silicone, additives, pigments, catalysts.

### Packaging

REF 9057MDR	2 Cartridges @ 50 ml 4 Mixing Tips
-------------	---------------------------------------