## »Temporary luting cement NE«

de

### Produktbeschreibung

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en-US

### Product description

*Temporary luting cement NE* is a temporary and eugenol-free luting cement that is easy to process. With its zinc oxide basis, it does not influence the curing of plastic-based filling and luting materials. *Temporary luting cement NE* is automatically mixed in the double-barreled syringe/cartridge and applied directly to the restoration or into the cavity. Manual mixing of the two components is not necessary.

### Indication

- Extensive tooth defects: Tooth decay, severe wear, traumatic injuries of the dental enamel
- Tooth loss

### Contraindication

Do not use the material where there are contact allergies or known allergies to any of the components.

### Intended purpose

Provisional luting of crowns, bridges, inlays and onlays.

### Limitations of use

Do not apply the material directly to the exposed dental pulp.

### Intended user

Dentist

### Patient target group

People treated in the course of a dental procedure.

### Recommended use

#### Preparing the preparation/restoration

1. Before cementing the surface, clean the surface of the teeth that are to be restored (e.g., with pumice stone powder or water).
2. Afterwards, rinse the prepared tooth and dry it with air. Avoid both excessive residual moisture and overdrying.

**Note:** The sensitivity of tooth cores can be reduced by applying a desensitizing agent (e.g., Fluoride Varnish). However, prior to final cementation completely remove such materials (without leaving any residue behind). All cement residue must be carefully removed when the temporary is subsequently removed. Residue of *Temporary luting cement NE* can be removed more easily if the cement is briefly chilled with cooling spray.

#### Inserting the restoration

1. Apply the *Temporary luting cement NE* directly to the restoration or into the cavity.
2. Apply gentle pressure to cement the restoration on the prepared tooth.
3. Ask the patient to slowly bite down (use cotton rolls if necessary) and check the occlusion.

### Removing excess material

1. After approx. 300 – 360 s, *Temporary luting cement NE* has hardened sufficiently so that excess material can be removed. Start by removing excess cement in the proximal areas.

**Note:** Residue of the cement can be removed more easily if it is briefly chilled with cooling spray.

2. Floss the interdental spaces.

### Timing

|   |             |
|---|-------------|
| Application on the provisional or the preparation and luting of the provisional on the prepared tooth | Within 30 s |
| Working time  | Max. 30 s   |
| Removal of any excess material  | 300 – 360 s |
| The temporary cement is fully hardened  | Max. 420 s  |

### Interactions

No interactions are currently known.

### Using the double-barreled syringe

The double-barreled syringes and cartridges enable the automatic dispensing and mixing, which allows a time-saving direct application. When using a new double-barreled syringe, remove the transport cap first. The mixing tip can then be fitted. After use and until the next treatment, leave the mixing tip on the double-barreled syringe as a seal. The used mixing tip should be replaced with a new tip when the device is used for the next treatment.

**Note:** When using a double-barreled syringe or cartridge for the first time, discard a small amount of the initially ejected material (e.g., a pea-sized amount).

### Cartridge handling

The double-barreled syringes and cartridges enable the automatic dispensing and mixing, which allows a time-saving direct application. When using a new cartridge, remove the transport cap first. The mixing tip can then be fitted. After use and until the next treatment, leave the mixing tip on the cartridge as a seal. The used mixing tip should be replaced with a new tip when the device is used for the next treatment.

**Note:** When using a double-barreled syringe or cartridge for the first time, discard a small amount of the initially ejected material (e.g., a pea-sized amount).

### Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

## »Temporary luting cement NE«

### Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.
- Product contains rosin

### Product performance characteristics

Complies with ISO 3107:2022, Type 1

### Storage/disposal

- Store in a dry place at room temperature (15 – 25 °C/59 – 77 °F)!
- Do not store in the refrigerator!
- Protect against exposure to direct sunlight!
- Leave the used mixing tip on the cartridge/syringe as a cap after application!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

### Composition

Zinc oxide, rosin, hydrogenated fatty acids, hydrogenated polydecene, maleic acid, carnauba wax, EBA, additives.

### Packaging

|             |  |
|-------------|--|
| REF 2037MDR | 2 Double-barreled syringes @ 5 ml,<br>20 Mixing tips |
| REF 2103MDR | 1 Cartridge @ 50 ml,<br>20 Mixing tips               |